

Intended Use:

The IBP cable is designed for the electrical connection between a patient monitor and a pressure transducer.

Structure:

Made up of plug, cable, transducer adaptor.

Storage and Handling:

When not in use, cables should be loosely coiled and stored in room temperature. Don't wrap cables around equipment to avoid damaging internal wires.

Operating Conditions:

Ambient temperature: 0 to +40°C

Relative humidity: 15 to 85%

Atmospheric pressure: 86kpa ~ 106kp

Cleaning and Disinfection:

The cable should be cleaned using a soft cloth moistened with soapy water or 70% isopropyl alcohol. Allow the cable to dry thoroughly before use. Do not use any abrasive agents or chemical product except 70% isopropyl alcohol.

Do not irradiate, autoclave, soak or immerse the cable in any kind of solution.

Transport: Use available means to transport. During transportation, should be taken to prevent extrusion and the rain or snow. Mixed consignment with poisonous, harmful and mordant product is prohibited.

Cautions: 1. Improper placement of the electrodes may cause inaccurate measurement.
2. Never immerse the cable assembly into the water or disinfectant.

Warnings: 1.Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
2.Do not attempt to reprocess, recondition or recycle the cables as these processes may damage the electrical components, potentially leading to patient harm.
3.Failure to properly connect the transducer or the cable to the patient monitor will result in intermittent reading, inaccurate results, or no readings.
4.To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
5.Always refer to the patient monitor operator's manual for complete instructions or additional instructions.
6.Carefully route cable and patient cable to reduce the possibility of patient entanglement or strangulation.

Contraindication: No

Dispose:

For disposable of accessories, follow local regulations or hospital's policy. Do not dispose randomly.

About Symbols:



Caution, consult instruction for use



Latex free



See Instructions for Use



Complies with the requirements of MDR EU 2017/745



LOT Number



Date of manufacture



MedNet EC-Rep GmbH

Borkstrasse 10, 48163 Münster, Germany

Manufacturer: Xuzhou Maicuff Technology Co., Ltd.

Address: 8#-2-1106 Jinxiujiayuan, Jianguo West Road, Xuzhou City, Jiangsu Province, China

Tel: 0086-516-85908810

Fax: 0086-516-85908830

<http://www.maicuff.com>

E-mail: info@maicuff.com