

**Intended Use:** NIBP Cuff is used with BP monitor to measure patient's arterial pressure.

**Structure:** NIBP Cuff contains Cuff, Bladder and Air Hose. Air hose has single tube and double tube.

**Product Code:**

Specification		Product Code
Reusable	Single, Nylon	M5101,M5102,M5103,M5104,M5104L,M5104XL,M5105,M5106
	Double, Nylon	M5201,M5202,M5203,M5204,M5204L,M5204XL,M5205,M5206
	Single, PU	M5111,M5112,M5113,M5114,M5114L,M5114XL,M5115,M5116
	Double, PU	M5211,M5212,M5213,M5214,M5214L,M5214XL,M5215,M5216
	Single/ Double, Polyester	M5121,M5122,M5123,M5124,M5124L,M5124XL,M5125,M5126 M5221,M5222,M5223,M5224,M5224L,M5224XL,M5225,M5226 M5133,M5134,M5134L,M5135,M5233,M5234,M5234L,M5235
	Single/ Double, Cotton	M5141,M5142,M5143,M5144,M5145,M5146 M5241,M5242,M5243,M5244,M5245,M5246
Reusable, Bladderless	Single	M5301,M5302,M5303,M5304,M5304L,M5305,M5306
	Double	M5401,M5402,M5403,M5404,M5404L,M5405,M5406
Disposable	Single/ Double	M5541-1#,M5541-2#,M5541-3#,M5541-4#,M5541-5#,M5542,M5543,M5544, M5641-1#,M5641-2#,M5641-3#,M5641-4#,M5641-5#,M5642,M5643,M5644, M5545,M5546,M5547,M5645,M5646,M5647

**Directions for Use:**

1. Choose correct cuff size according to patient's limb circumference and monitor.
2. Refer to the figure below to apply the side of "This side to patient" on the cuff to patient's arm or thigh and Symbol " ↓ " or "  $\phi$  " on the cuff is over the brachial or femoral artery, wrap the cuff around the arm or thigh.



3. Connect the cuff with NIBP air hose.

**Cleaning and Disinfection (Only for Reusable NIBP Cuff):**

1. To clean, the cuff can be wiped with a damp cloth with mild soap water.
2. The cuff can be disinfected with ultraviolet.

**When to Replace Cuff:** Please replace the cuff timely when it's damaged. If the monitor displays to leak, replace it too.

**Storage Requirement:** Temperature: -5℃-40℃ Humidity: 0%-85% Atmospheric: 86Kpa-100Kpa

**Transport:** Can use available means transportation for transport. During transportation, should be taken to prevent extrusion and the rain or snow. Mixed consignment with poisonous, harmful and mordant product is prohibited.

**Caution:** Placed improperly may cause inaccurate measure. Single patient multiple used, discard after use and cannot be used for other patients.

**Warnings :** Don't place the Cuff on the limb of being used for intravenous infusion or Spo2 measurement.

**Contraindication:** No

**About Symbol on the Cuff or Label**

Refer to instruction manual	Manufacturer
The cuffs don't contain nature rubber latex	Unique Device Identifier
Single patient multiple used	Reusable Do not throw away
Medical Device	Authorized Representative in European
The cuffs comply with the requirements of MDR EU 2017/745( Medical Device Regulation)	



**MedNet EC-Rep GmbH**

Borkstrasse 10, 48163 Münster, Germany

**Manufacturer: XUZHOU MAICUFF TECHNOLOGY CO., LTD.**

Address: 8#-2-1106 Jinxiujiayuan, Jianguo West Road, 221006 Xuzhou City,

Jiangsu Province, PEOPLE'S REPUBLIC OF CHINA

Tel: 0086-516-85908810

Fax: 0086-516-85908830

http://www.maicuff.com

E-mail:info@maicuff.com

